# ADDITION CURING SILICONES

# ORMAPLUS<sup>®</sup> PUTTY

## SOFT CONSISTENCY

### FAST SETTING - REGULAR SETTING

**Ormaplus<sup>®</sup> Putty Soft** providing a reduced final hardness (60 Shore-A) indicated for problematic clinical situations (patients with periodontal compromised residual dentition).

**Ormaplus<sup>®</sup> Putty Soft** is available in two versions:

- **Fast:** setting time 4' 00"
- **Regular:** setting time 5' 30"

Polyvinyl siloxane based, addition curing, elastomeric precision preliminary impression material.  
Final hardness: 60 Shore-A

**Ormaplus<sup>®</sup> Putty Soft** is designed to provide the highest precision in details reproduction and a perfect chemical bonding to light-bodied impression materials.

**Ormaplus<sup>®</sup> Putty Soft** can be kneaded smoothly and can be easily removed from patient's mouth thanks to its elastic properties.

**Ormaplus<sup>®</sup> Putty Soft** formulation guarantees high tear resistance and compressive strength combined with an excellent elastic recovery.

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# ORMAPLUS<sup>®</sup> LV

## FAST SETTING - REGULAR SETTING

Polyvinyl siloxane based, low viscosity, addition-curing material, for second precision impression.

Ormaplus<sup>®</sup> LV is available in two versions:

- **Fast setting** (indicated for the "two-step putty/wash" technique) providing a reduced intraoral setting time (2' 30") for a better patient's comfort.
- **Regular setting** (indicated for the "one-step putty/wash" technique) providing a standard intraoral setting time (4').

Ormaplus<sup>®</sup> LV is a thixotropic impression material for putty/wash (one step and two step), sandwich and relin impressions.

Due to the use of new surface-active components in Ormaplus<sup>®</sup> LV formulation, the hydrophilicity of the product is more marked and leads to the realization of accurately detailed impressions even in presence of oral fluids.

Thanks to the high tear-resistance of Ormaplus<sup>®</sup> LV the interdental septae reproductions are not ripped while removing the impression from the mouth or while casting the model.

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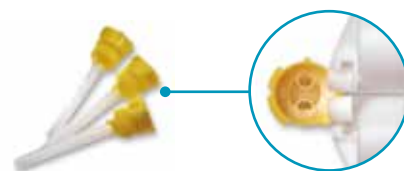
**ENGLISH**
**ORMAPLUS®**  
 System  
 Main features

- Putty materials mixing and light bodied materials dispensing ease.
- Perfect chemical bonding between putty and light bodied materials.
- High hydrophilicity and tear resistance.
- Easy disinfection without dimensional alterations.
- Permanent dimensional stability.
- Perfect compatibility with any kind of plaster (natural or synthetic).
- Possibility of casting the same impression several times.
- Odourless and tasteless material for a better patient's comfort.
- Cartridges with separated dispensin

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